

Deafgard Quick Start User Guide



Keypad

- Button 1 - Power On/Off
- Button 2 - Time Set
- Button 3 - Increment
- Button 4 - Mute/Snooze
- Button 5 - Alarm Set
- Button 6 - Alarm On/Off

How to Use

1. Ensure unit is On (listening circuits will not work if unit is Off).
2. Place Pillow Pad under your pillow, with the Deafgard unit on the bedside table.
3. When a fire alarm is detected the Pillow Pad will vibrate constantly, the Strobe will flash and the LCD will display the message "FIRE".

NOTE: The Pillow Pad will not vibrate if placed in the Cradle.

Setting Time

1. Hold Button 2 for two seconds.
2. Use Button 3 to adjust hour.
3. Press Button 2 to save hour.
4. Use Button 3 to adjust minute.
5. Button 2 will save set time.

NOTE: Deafgard uses a 24 hour clock.

Setting Alarm Clock

1. Hold Button 5 for two seconds.
2. Use Button 3 to adjust hour.
3. Press Button 5 to save hour.
4. Use Button 3 to adjust minute.
5. Button 5 will save Alarm time.
6. Press Button 6 to turn the Alarm Clock On (a bell icon will appear on the LCD).
7. Button 4 will Snooze the Alarm for eight minutes before repeating the alarm.

NOTE: When the Alarm is sounding the Pillow Pad will pulse and the Strobe will not flash.

To stop Alarm whilst it is sounding/
Pillow Pad is vibrating press Button 6.

Refer to Operating Guide for detailed instructions.